Liberty® Mobilgrip 500

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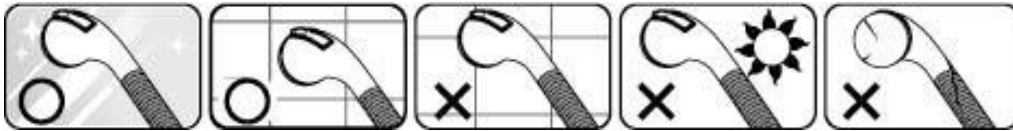
WARNING: *Be sure to read all information and directions before use!!*

- ▲ The suction cup grab bar is not designed to hold or support your full body weight.
- ▲ This grab bar is designed to assist your natural motion, as when stepping into or out of a shower.
- ▲ Always check the attachment and holding condition of suction cup through windows every time before each use.
- ▲ Do not attach to moving surfaces.
- ▲ Maximum User Weight : 140 lbs (Lateral&Vertical); 70 lbs (Angled)

It is designed for use only on smooth and non-porous surfaces; tile, glass, porcelain or fiberglass. Not for use on wall board, wood, painted or papered surfaces!!

FITTING:

1. Thoroughly clean the surface with alcohol wipe and allow to dry and check the condition of the suction cups. Do not use if damaged, cracked or split.
2. Raise both levers, place the Liberty® Suction Grab Bar 500mm so that both suction cups lie firmly and completely on a smooth surface.
3. Hold one end and press on one lever with your palm, You should feel resistance as the suction cup tightens.

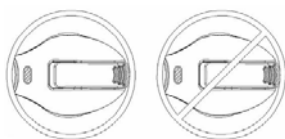
REMOVAL:

1. Lift each of the two levers.
2. If suction remains, gently lift one edge with a finger.
3. Do not use a sharp instrument which may damage the suction cup.
4. Always keep the levers flip up until next use.

PATENTED SUCTION INDICATOR

"GREEN" means the correct attaching, otherwise, it will turn into the "RED" request to check the mounting surface and re-attach again. Do not place over joints or grout lines.

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Rehaforum Medical GmbH