Mamma Pads

THERALINE

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© 2024 Theraline

EN MAMMA PADS - THE VIRTUALLY INVISIBLE BREAST FEEDING INSERT

We are pleased that you decided on Mamma Pads from Theraline. Our breast feeding inserts prevent breast milk from running out by applying gentle pressure and thus use a natural reflex of the female breast. Mamma Pads permanently provide careful protection to you during the entire nursing period in a natural way.

The breathable and thus skin-compatible silicone pads are comfortable, can be used for up to several months and afford reliable comfort without being noticeable. Mamma Pads can accompany you during your whole nursing phase and remain virtually almost invisible unlike conventional breast feeding inserts. Wear Mamma Pads comfortably under your clothes day and night, or alternating with conventional breast feeding inserts, when going out, during sport or even when swimming. Mamma Pads stay in place completely by themselves.

Operating instructions

Note: For hygienic reasons we recommend that the Mamma Pads first undergo a boiling wash for approx. 1-2 minutes before use without the protective cups.

First turn the adhesive side of the Mamma Pads outwards and then press it dead centre onto the nipple. It is important in this case that the nipple is carefully pushed with the finger, in order to achieve the desired effect. No milk leaks out of the breast when the nipple is pushed in.

Afterwards, the Mamma Pads can be smoothed out carefully with the hand onto the breasts and will now provide you with special protection from this time on. After removing the Pads, they should be cleaned (see cleaning instructions) and be stored together with the protective cups in the storage box.

Note: Fatty skin decreases the adhesive power of the Mamma Pads. For this reason, always ensure that your breasts are clean and dry before applying, in order to guarantee a good service life. The hands should also be thoroughly washed, before you separate the Mamma Pads from their protective cups and apply.

- **Non-absorbing, breathable**
- **Prevent breast milk from leaking out using gentle pressure**
- **Comfortable and skin-compatible**
- **May be used up to several months**
- **Sleep without a bra**
- **Not visible e.g. under a T-shirt, evening gown or bathing suit**

Application

The breast feeding inserts made of breathable silicone prevent the escape of breast milk, are re-usable and virtually invisible. Contrary to conventional breast feeding inserts, you will not be restricted in everyday activities and can rely on their protection in all circumstances. Even during sport, swimming (breast must be dry before fitting the Mamma Pads) or when going out in the evening, Mamma Pads work reliably without being noticeable.

Contraindications

Do not use Mamma Pads on inflamed and sore nipples, as well as on sunken and flat nipples. Furthermore, we advise against use during pregnancy. If irritation, skin changes (e.g. redness) or similar should arise, we recommend that their use be interrupted and to consult your breast feeding counsellor, midwife or physician.

Note

If the pads are not carefully attached or have reduced adhesion - e.g. with insufficient cleaning or at the end of the product's useful life, as well as during sporting activities associated with strong sweating - small pools of milk can form, from which a larger quantity of milk may be released at one time. During breast feeding and in particular with mothers, who have a very strong flow of milk, it is recommended to cover the non-feeding breast with a conventional breast feeding insert.

Cleaning: Clean Mamma Pads each time after use with warm water and pH-neutral soap (without lotions and oils) and leave them to dry afterwards in the air.

Duration of use: Daily, also at night. With sensitive nipples and a high breast feeding frequency, alternating use with conventional breast feeding inserts is recommended.

Durability: Depending upon the level of care and use, up to several months. Material discoloration during production does not impair the function and the durability of the Mamma Pads. Note: Generally, do not rub the adhering surface!

Cleaning: Clean the Mamma Pads after every instance of use with warm water and pH-neutral soap (without lotions or oils) and then leave to dry in the air. Warning: do not dry product with a towel! Fibrous materials shorten the lifespan of your Mamma Pads!

Duration of application: Daily, also nights. If you have sensitive nipples or a high breastfeeding frequency, changing over to conventional breastfeeding pads is recommended. Tip for sensitive nipples: put a very small amount of cream exactly on the nipples to prevent adhesion. Theraline recommends Lansinoh breastfeeding pads and Lanolin ointment are used in combination with the Mamma Pads.

Durability: Up to several months depending on care and use.

Warning: Do not use product when suffering from inflamed or sore nipples. If irritations, changes to the skin (e.g. reddening) or similar should occur, we recommend you to stop using the product and to consult your midwife or your doctor. To prevent the risk of suffocation store the Mamma Pads away from infants in their packaging!

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