MATERNITY

REF

## Maternity belt

1708, 1710


Gebrauchsanweisung. Instructions for use. Návod k použití. Bruksanvisning. Kasutusjuhend. Instrucciones de uso. Käyttöohje. Mode d'emploi. Upute za uporabu. Használati utasítás. Istruzioni per l'uso. Naudojimo instrukcija. Lietošanas pamācība. Arahan penggunaan. Gebruiksaanwijzing. Bruksanvisning. Sposób użycia. Manual de instruções. Instrucțiuni de folosire. Bruksanvisning. Navodilo za uporabo. Návod na použitie. Упътване за употреба. 使用说明. Οδηγίες χρήσης. 取扱説明書. 사용 지침. Инструкция пользователя. Інструкція з використання คำแนะนำสำหรับการใช้งาน. داخلت سالانہ امی لعت

DE .....	3	GR .....	14	PL .....	25
EN .....	4	HR .....	15	PT .....	26
AR .....	5	HU .....	16	RO .....	27
BG .....	6	IT .....	17	RU .....	28
CN .....	7	JP .....	18	SE .....	29
CZ .....	8	KR .....	19	SI .....	30
DK .....	9	LT .....	20	SK .....	31
EE .....	10	LV .....	21	TH .....	32
ES .....	11	MS .....	22	UA .....	33
FI .....	12	NL .....	23		
FR .....	13	NO .....	24		

**REF**

Maternity belt  
1708, 1710

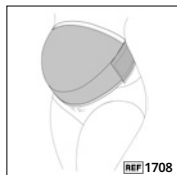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## Instructions for use

All product-related serious incidents must be reported to the manufacturer of ANITA, Dr. Helbig GmbH, as well as the responsible authority.

### Article

1708 Circular pregnancy bandage

1710 Circular pregnancy bandage with reinforcing elements

### Indication

Target group: Women during pregnancy.

Pregnancy bandage to support the belly during pregnancy.

- For pelvic girdle pain
- For lumbar back pain
- After childbirth to stabilise the abdominal wall

### Contraindications and risks

- In case of skin diseases (skin changes)
- In case of lymphatic drainage, circulatory and sensory disorders in the body region to be treated

In rare cases, allergic reactions may occur. Please contact your doctor or healthcare professional if you are not sure whether you may be allergic, or if you have questions or issues regarding the use of these products.

### Safety information

- If numbness occurs, loosen the medical product or remove it, if necessary.
- The bandages must not be worn on open wounds in the abdominal, pelvic and back area.

### Purpose/How it works

The circular bandages are used for abdominal support and relief in the back and frontal torso area during pregnancy. As the tension can be individually adjusted, pain caused by the increasing weight of the belly

can be alleviated.

The hook-and-loop fastening can be adjusted seamlessly and individually; it can therefore be used throughout pregnancy and after childbirth.

### Materials

- Cotton, polyester, polyamide, elastane (latex-free)
- See sewn-in label for material composition percentages
- ANITA products bear the OEKO-TEX® STANDARD 100, No. 98.0.1439 Hohenstein HTTI mark. It guarantees the absence of harmful or allergenic substances.

### Care instructions

- Please refer to the sewn-in label for care instructions
- Close hook-and-loop fasteners before washing
- Washing the bandage in a laundry bag is recommended

### User instructions

Allow a trained professional to select the correct size for you.

1. Opening the pregnancy bandage: Loosen the hook-and-loop fasteners
2. Position the wide surface of the bandage at the height of the lower back and close it by placing the right end over the left end below the stomach.
  - 1710: Make sure that the Anita Logo is not upside down and is legible (see Fig.).
3. Close the hook-and-loop fastener as tightly as you find comfortable.
  - 1710: Grip both side elastic bands at the same time and tighten them as evenly as possible. Attach the bands to the hook-and-loop fastenings provided.

If the bandages are too tight or too loose, repeat steps 1–3 and vary the tension of the elastic bands.

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ANITA Dr. Helbig GmbH, Grafenstraße 23, 83098 Brannenburg, DE, [www.anita.com](http://www.anita.com)
Corsa Nova AG/SA Anita Group, Wassergrabe 14, 6210 Sursee, CH
ANITA UK Ltd., 23 Shirwell Crescent, Furzton, Milton Keynes, MK4 1GA, GB