Imported by - Importé par:
GD import (Genius Ideas & Wellys)

190 chemin de la Frayère - 06530 Peymeinade - France

www.gdimport.com

www.genius-ideas.com

www.wellys-care.com

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GB – HANDY EAR CLEANER "SYRINGE"– 013930

1. Opening for soft tip.
2. Plunger.
3. Soft tips.
4. Water working position.

Hygienic, practical and comfortable, this special syringe is essential for cleaning the ear. No risk of pushing the earwax inside the ear canal. The conical head allows a triple spray and cleans the walls of the auditory canal without aiming at the eardrum. For a painless and safe ears cleaning on children and adults!

Capacity: 20ml. Delivered with 3 silicon tips.

CAUTION:

- Clean before and after each use.
- Not for use on children under 3 years of age.
- If you experience any pain or dizziness, discontinue use.
- Never irrigate ears if eardrum is known to be perforated or if ear canal is bleeding, drainage, or if pain is present.
- Stop use in case of skin problem and consult a specialist.

USE INSTRUCTIONS:

1. Screw the soft tip on the opening at the end of the syringe until it is securely placed.
2. Press the plunger completely into the syringe.
3. Dip the tip into warm water (at body's temperature) and pull the plunger out to fill the syringe.
4. In a seated position, tilt your head sideward and place a bowl under the ear to catch excess water. Slowly press the plunger down.
5. You may need to repeat the process a few times.

CLEANING:

- Wash syringe with warm soapy water after each use, rinse and let it air dry.
- Syringe should be stored in a cool and dry place.

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