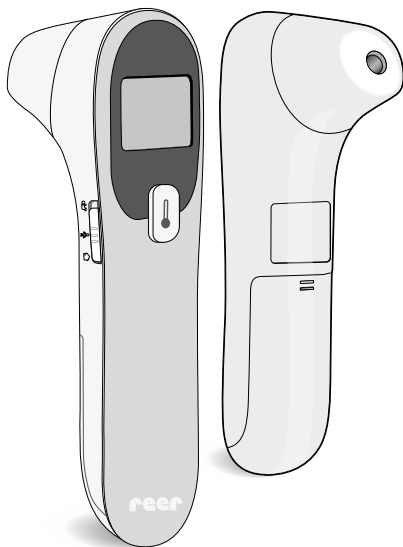
4in1 Colour MaxTemp

Infrarot-Fieberthermometer

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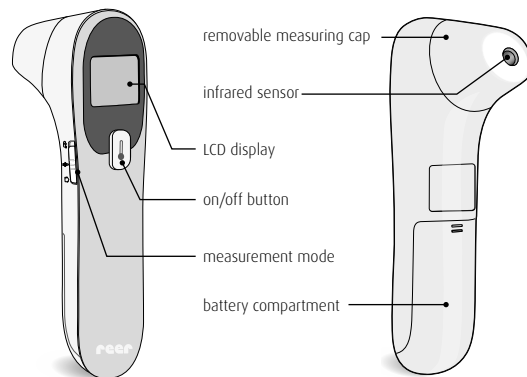
4in1 Colour MaxTemp Infrared clinical thermometer

The innovative reer thermometer 4in1 Colour MaxTemp lets you determine the body temperature in the ear and on the forehead reliably and within seconds. Contactless temperature measurement guarantees a safe and hygienic measurement process and reduces the risk of transmission of viruses or other pathogens.


A backlit display and the option to silence the signal tone allows you to measure your child's temperature during the night without waking them.

**IMPORTANT!
READ CAREFULLY BEFORE USE
RETAIN FOR FUTURE REFERENCE.**

Device description

Display

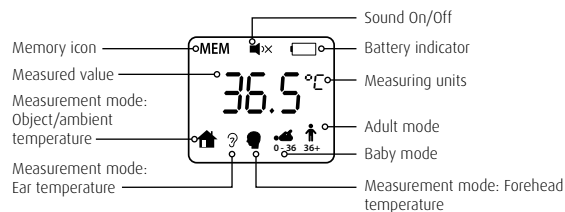Technical data

Dimensions:	143 x 38 x 51 mm
Material:	Plastic
Output/nominal voltage:	0,27W / 3V
Expected lifetime of thermometer:	5 years
Type of battery:	2 x 1,5V AAA batteries (included in delivery)
Battery life:	Approx. 2.300 measurements
Measuring range:	
Body temperature:	32,0 °C – 43 °C (89,6 °F – 109,4 °F)
(Forehead/ear)	
Object/ ambient temperature:	0 °C – 99,9 °C (32,0 °F – 211,8 °F)
Measuring accuracy:	
Body temperature:	±0,2°C (±0,4°F)
Object temperature:	± 1°C (± 2°F)
Measuring units:	Celsius (°C) and Fahrenheit (°F)
Measuring point:	on the forehead / in the ear
Measuring time:	approx. 1 second
Measured value memory:	25 measurements
Measuring display:	Liquid crystal display (LCD)
Acoustic signal:	switchable on/off
Automatic switch-off:	approx. 13 seconds following last measurement
Net weight:	approx. 85g
Protection class:	IP22
Operating conditions:	10°C – 40°C (50°F – 104°F) ambient temperature, 15% – 95% relative humidity, 70 – 106 KPa air pressure
Storage conditions:	-25°C – 55°C (-13°F – 131°F) ambient temperature, 15% – 95% relative humidity

Safety instructions

Please read the following points through carefully and follow the instructions before using the device.

Danger

- ! Choking hazard! Keep the battery out of the reach of children.
- ! If batteries have leaked, remove using chemical resistant protective gloves and protective glasses.
- ! In case of high fever or unclear measurement results, always seek advice from a doctor.
- ! The infrared technology your reer thermometer uses is perfectly safe. Although it's not harmful to look through the lens while the temperature is being measured, this is not recommended.

Warning

- ! Check the device for visible damage before each use. If you detect any damage, do not use the device.
- ! The minimum measurement time until the signal tone sounds must always be observed.
- ! Avoid extreme heat and direct sunlight.
- ! Do not subject the device to mechanical shocks.
- ! With the exception of the cover on the battery compartment, the device must not be opened as this can lead to electric shocks. If a device is malfunctioning, please do NOT try to fix the problem yourself. Please contact the retailer in this case.
- ! The unit is not waterproof and must therefore not be submerged in water or directly exposed to any liquids.
- ! Do not submerge the device in water, other fluids or flammable detergents, in order to avoid the risk of an electric shock and/or the danger of fire.

Caution

- ! Please only use the device for its intended purpose.
- ! Only use the device for private use, it is not intended for commercial use.
- ! This product is not a toy! Do not allow children to play with the device.
- ! Do not use the device outdoors.
- ! Remove all packaging material before use.
- ! Protect the measurement sensor's lens from dirt or damage.
- ! The measured core body temperature (measured rectally) can be significantly higher than the surface temperature (measured at the temple), depending on the course of the fever.
- ! A measurement of the core body temperature using a rectal clinical thermometer is recommended:
 - if the measurement result is surprisingly low
 - for newborns within the first 100 days
 - for children under the age of 3 who have a compromised immune system or if a fever does or does not occur where a fever would or would not be expected.

Scope of delivery

- 1 x thermometer
- 2 x AAA alkaline batteries
- 1 x instruction manual

Before first use

Before using your thermometer, make sure that the sensor is clean. A dirty sensor can lead to incorrect measurement results.

Inserting batteries

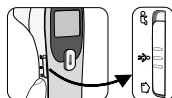
The batteries must be inserted into the thermometer before it can be used.

1. Open the battery compartment on the underside of the device.
2. Insert the batteries.
3. Close the battery compartment.

Note: Do not use different battery types or combine new and used batteries. Batteries must be inserted with the correct polarity.


Measurement of body temperature

The thermometer has been designed to measure the body temperature on the forehead or in the ear. If the measurement is taken from another part of the body, this may result in inaccurate measurement data.


Set the control on the side to the desired measurement mode. Here you have the choice between:

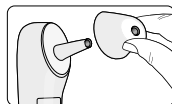- the baby mode (0 – 36 months)
- the adult mode (from 36 months)

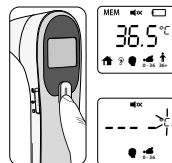Remove the measurement cap, before taking the temperature in the ear.

Switch the thermometer on by briefly pressing the On/Off button.
The backlight and start sequence are displayed.
The display shows a dashed line with the flashing temperature unit.

Position the thermometer a maximum of 3 cm from the forehead or the ear. Start the measurement with the On/Off button. After the signal tone, the temperature can be read immediately.

Notes on measuring body temperature

- Before each measurement, make sure that the sensor is clean. If this is not the case, please clean the sensor according to the instructions on page 20.
- Vasoconstrictive drugs can influence the measured value.
- Portable and mobile communication devices can also affect the device.
- Wait at least 30 minutes after any sporting activities, bathing or eating before taking a measurement.
- If possible, do not use the thermometer outdoors, but only in closed rooms. External influences such as wind and outdoor temperature can influence the measurement result.
- If the device is to be used in an extreme-temperature environment, store it at room temperature for at least an hour before taking a measurement.
- Holding the thermometer in the hand for too long before taking the measurement will cause the device to heat up. This can cause a false measurement result to be given.

Fever alarm

The thermometer has a built-in optical indicator to warn you in case of elevated body temperature. The coloured display shows you the temperature range that is to be estimated.

Baby temp. mode	Adult temp. mode	Display	Measured body temperature	Signal tone
< 32,0°C (89,6°F)	< 32,0°C (89,6°F)	Lo	too low	1 x long
32,0°C – 37,2°C (89,6°F – 99,0°F)	32,0°C – 37,4°C (89,6°F – 99,3°F)	green	normal	1 x long
37,3°C – 37,9°C (99,1°F – 100,2°F)	37,5°C – 37,9°C (99,5°F – 100,2°F)	orange	elevated	1 x long
38,0°C – 43,0°C (100,4°F – 109,4°F)	38,0°C – 43,0°C / (100,4°F – 109,4°F)	red	high	6 x short
> 43,0°C (109,4°F)	> 43,0°C (109,4°F)	Hi	too high	6 x short

! If the device's LED lights up red during temperature measurement, the person's body temperature should be constantly monitored and where there is uncertainty a doctor's advice should be sought.

Use of the device is in no way a substitute for a doctor's examination. In case of high fever or unclear measurement results, always seek advice from a doctor.

Measuring object / ambient temperature

If possible, the measurement of liquids should take place from outside the vessel to avoid any condensation forming on the sensor. The fever alarm is switched off in this mode. The display is constantly lit green.

The object/ambient temperature mode shows only the exposure temperature of the surface/room temperature, and does not provide any indication of body temperature. This measurement mode cannot therefore be used to measure fever.

Set the control to object measurement . Follow steps 1 to 3 as for body temperature measurement. Point the thermometer at the desired object and press the measure button. Read the measured temperature on the display.

Setting functions

Sound mode: select signal tone or mute

	1. Make sure that the thermometer is switched off.
	2. Press and hold the On/Off button for approx. 12 seconds , until the display shows „ON“ or „OFF“.
	3. Press the On/Off button again in order to select the mode.
	4. Press and hold the On/Off button again for approx. 3 seconds in order to save your selection.

Memory function

	1. Make sure that the thermometer is switched off.
	2. Press and hold the On/Off button for approx. 5 seconds until the display reads „MEM“.
	3. Press the On/Off button to click through the last 25 stored values.
	4. The thermometer starts with the last measured temperature at number 1.
	5. When the memory is full, the oldest value is automatically deleted and the newest value is saved.
	6. The display lights up in the respective display colours of green, orange and red, as long as it is a body temperature measurement that is being displayed.
	7. To exit from the menu item, let the thermometer rest for approx. 10 seconds until it automatically switches off.
	8. To erase the memory, remove the batteries.
	9. Press and hold the On/Off button while simultaneously replacing the batteries, until the display shows „---“.

Change temperature scale

	1. Make sure that the thermometer is switched off.
	2. Press and hold the On/Off button for approx. 8 seconds until the display shows „C“ or „F“.
	3. Press the On/Off button again in order to choose the units of temperature measurement.
	4. Press and hold the On/Off button again for approx. 3 seconds in order to save your selection.

Troubleshooting

Problem/symbol	Possible cause	Action
Unit won't switch on.	Batteries are dead.	Insert new batteries.
	Batteries are wrongly inserted.	Check the position and polarity of the batteries.
Incorrect or fluctuating temperature values	Sensor dirty/ broken	Check the measurement sensor and clean it if necessary.
	Low battery level, measurements are still possible.	Replace the batteries soon.
	Measured body temperature exceeds 43 °C (109,4 °F).	Check to see whether the correct measurement mode is active (body temperature / object temperature). Repeat the measurement. In doing so, make sure the measurement sensor is correctly positioned.
	Measured object/ ambient temperature exceeds 100 °C (212 °F).	
	Measured body temperature is below 32 °C (89,6 °F).	Check to see whether the right measurement mode is active (body temperature / object temperature). Repeat the measurement. In doing so, make sure the measurement sensor is correctly positioned.
	Measured object/ ambient temperature is below 0 °C (3 °F).	

Cleaning and care advice

The sensor is the most sensitive part of the infrared thermometer. Make sure not to touch or scratch the surface of the measurement sensor. Should it become dirty, carefully and gently rub the sensor using a cotton bud soaked in alcohol or mild cleaning agent (e.g. very diluted washing-up liquid). Do not use any aggressive cleaning agents on the sensor. Clean the housing and display using a soft, dry cloth or an alcohol wipe. The device is not waterproof and so should not be immersed in water or exposed directly to moisture. Remove the batteries from the device if it is to be out of use for a longer period. This product does not require any special maintenance. Repairs to the device must only be carried out by a qualified professional. Keep the thermometer dry, dust-free, away from dirt and direct sunlight.

Metrological control

As a rule, checking the thermometer's precision is not required.

For devices used in Germany for medical practice, the Medical Products Operator Ordinance of 29 June 1998 (in the version announced on 21 August 2002, last amended by Article 4 of the law of 29 July 2009) requires metrological control after 2 years. Gauging offices provide paid metrological inspections.

Legal basis for placing on the market in the EU

The manufacturer has been certified by the EU notified body TÜV SÜD Produkt Service GmbH in line with the 93/42/EEC Directive on Medical Devices.

Legend

The following symbols are used in the user manual, on the packaging and on the type plate of the device and the accessory:

	Caution
	Permitted storage and transport temperature and air humidity
	Permitted operating temperature and air humidity
	Manufacturer
	Observe the user manual
	Application part type BF
	Disposal according to the Waste Electrical and Electronic Equipment EC directive
	Batch designation
	Protection against entry of solid foreign bodies with a diameter of >12.5 mm. Protection against dripping water with a 15° incline.
	The CE marking certifies conformity with the basic requirements of the 93/42/EEC guidelines for medical products.

Warranty

Please retain your receipt, the user manual and item number. Warranty claims or claims from promises of guarantee must be processed by the dealer. The technical specification, information and characteristics of the product described herein have been compiled to the best of our knowledge and belief and are correct at the time of printing. Any information provided by reer should be correct and authoritative. Nevertheless, reer does not accept liability for potential errors in this document and reserves the right to make changes to the product design and/or specifications without prior notice. Reproduction and disclosure of information contained in this copyrightprotected document, in any form or by any means - graphical, electronic or mechanical by photocopying, recording on tape or storage in a data retrieval system (even in part), require prior written consent from reer.

Disposal instructions

At the end of its useful life, this product must not be disposed of with normal household waste. Owners of old devices have a duty to hand them in to a collection point for the recycling of electronic devices. Please contact your local council to find out more about an appropriate disposal centre. It may be possible to hand in small devices with a maximum edge length of less than 25 cm at an electrical retailer. The relevant symbol on the product, in the user manual or on the packaging refers to these duties.

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SHENZHEN AEON TECHNOLOGY CO., LTD. RM6H02, BLOCK 27-29,
reer GmbH • Mühlstr. 41 • 71229 Leonberg • Germany • www.reer.de