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Cleaning – CAUTION!

Do not use any caustic or abrasive cleaning agents or sponges. These can damage the surface.

Disposal

Dispose of the packing in an environmentally friendly manner. Dispose of it where valuable waste is collected. Dispose of the product through an approved disposal company or the municipal disposal facilities. Observe the current regulations. If in doubt, consult your local authority about an environmentally friendly disposal.

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EN INSTRUCTIONS – BED RAIL

Dear customer,

Your new bed rail offers optimum functionality and comfort. We wish you much enjoyment with it. Read these instructions carefully before assembling and using the product! Retain the instructions for future reference! If you pass the product on, please also pass on these instructions. The manufacturer and importer assume no liability for incorrect use of the product or non-observance of the information in these instructions!

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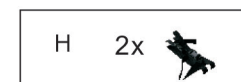
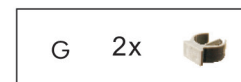
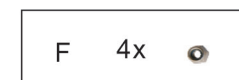
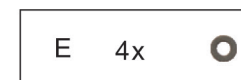
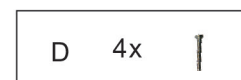
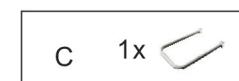
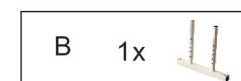
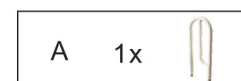
List of part and contents

A Supporting handle
B Adjusting tube
C U tube
D Screw
E Washer
F Nuts
G Lock pin
H Strap

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Safety Instructions

- Keep the packing material away from children (risk of suffocation).
- Keep children away from the bed rail. They could attempt to stand on it or pull themselves and fall over.
- Ensure that a another person is close by. This person can help you with sitting down or standing up if necessary.
- Inspect the bed rail before use and ensure that it is fastened securely.
- Bed rail is designed to assist your natural motion, it can't support your full body weight!
- The bed rail supports max. 65 kg.

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Assemble

1. Install the supporting handle (A) to the adjusting tube (B).
2. Fix adjusting tube (B) to U tube (C), tighten the screw and nuts (D) + (F). Place the washer (E) under the screw (D).
3. Adjust the handle height by (A) + (B). Ensure the lock (G) is totally insert hole.
4. Fasten the strap (H) with the bed tightly.

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